

INDICATION

RADIESSE injectable implant is FDA-approved for subdermal implantation for the correction of moderate to severe facial wrinkles and folds, such as nasolabial folds.

IMPORTANT SAFETY INFORMATION FOR RADIESSE

Who should not use RADIESSE?

RADIESSE should not be used in people who have an allergy to any component of the product. You should not use RADIESSE if you are pregnant or breast feeding or if you are under 18 years of age.

What is the most important information I should know about RADIESSE?

As with all procedures that involve an injection through the skin, there is a risk of infection. Do not use RADIESSE if you have a skin infection until it has healed.

The microspheres in RADIESSE can be seen in X-rays and CT Scans. It is very important that you tell your health care provider that you have had RADIESSE injected into your face.

If you have a history of herpes, you may experience a herpes breakout after receiving RADIESSE.

For approximately 24 hours after treatment:

- Avoid significant movement or massage of the treated area
- Do not apply makeup
- Avoid extensive sun or heat exposure

What should I tell my doctor before being treated with RADIESSE?

Tell your health care provider if you are taking blood thinners or medicines that can interfere with the clotting of blood, such as aspirin or warfarin. These medicines might make it more likely that you have bruising or bleeding at the injection site

What are the most common adverse events with RADIESSE?

The most common adverse events include bruising, redness, swelling, pain, tenderness and itching. These side effects generally last for a short time and are mild in nature.

These are not all of the possible side effects with RADIESSE. Tell your health care provider about any side effects that bother you or do not go away.

Important: For complete instructions for use, please visit www.Radiesse.com or call Merz Aesthetics Customer Service at 866-862-1211

Caution: Rx Only